

## easy-slide

another Arion brand



Slide the Easy-Slide over the foot

1




Slide the stocking over the Easy-Slide

2




Draw the stocking over the heel

3

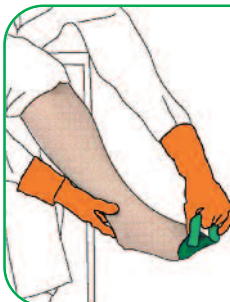

Place one hand firmly on the heel and with the other hand pull the loop of the Easy-Slide

4

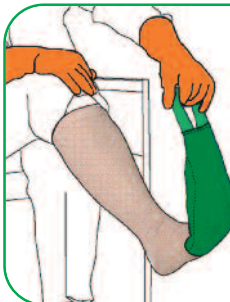

Pull the Easy-Slide out in small movements

5




Slide Easy-Off  
over the stocking

1




Fasten the velcro  
strap firmly just  
above the ankle

2

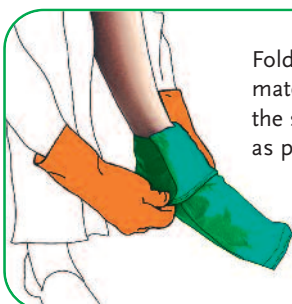

Fold the spare  
material down over  
the strap as far  
as possible

3

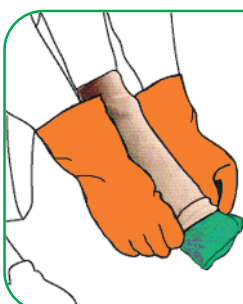

Fold the stocking  
over the Easy-Off  
as far as possible

4



With small  
movements, starting  
at the heel, slide the  
stocking and  
Easy-Off over the heel

5